

Tunneller

A New Flexibility in Shunt Surgery




AESCULAP®

Tunneller

Catheter Tunnelling Instrument

- Tunnelling instrument in eight variants for individual surgical requirements
- Handle customizable to personal preference
- Each tunneller has a blunt and sharp tip, which makes it adaptable to the tissue at hand
- Designed for easy and fast (direct) catheter placement
- Bi-directional approach possible: paraumbilical → retroauricular
retroauricular → paraumbilical


HANDLING OPTIONS

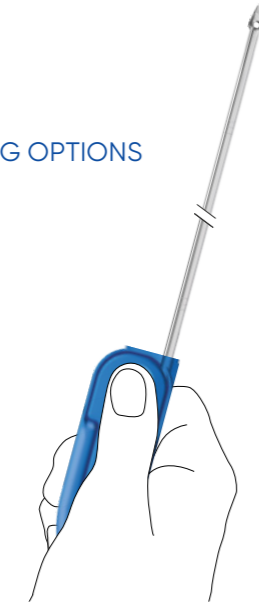


Step 1 - Longitudinal alignment



Step 2 - Press lever and remove shaft

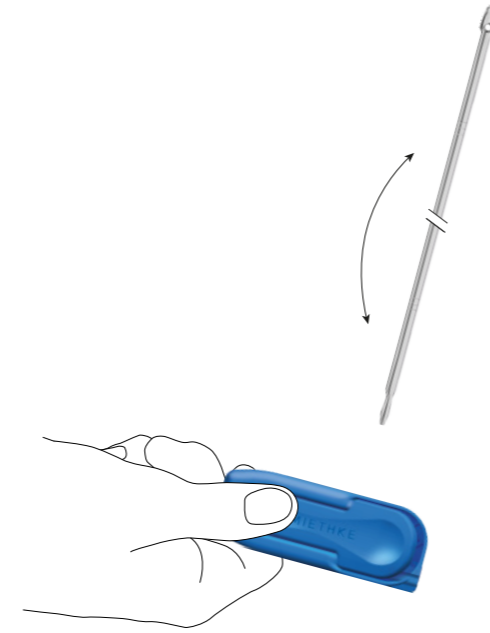


Step 3 - Choose appropriate tip design and handle alignment (here lateral)

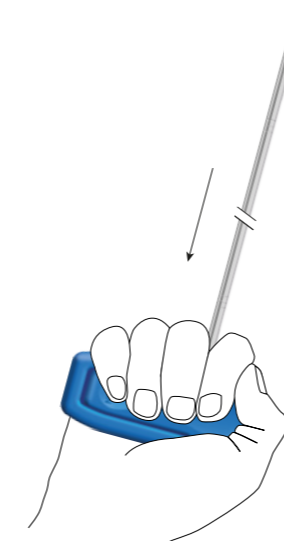


Step 4 - Close upper handle part to lock shaft

TUNNELLING SHAFT

- 2 options: standard and rigid
- Sharp and blunt tip
- Shaft tip can be selected intraoperatively
- Shaft designed to be easily shapable
- Solid material without buckling behaviour
- Single piece rod design - no random components
- Opening for catheter insertion for direct loading
- Flattened tip design for controlled advancement
- 4 lengths: 300, 450, 600, 700 mm

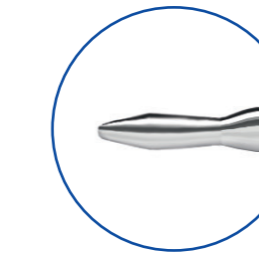



SHARP TIP: designed for solid tissue / fascia

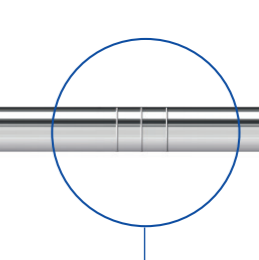

Standard version with ring marking



BLUNT TIP: designed for soft tissue

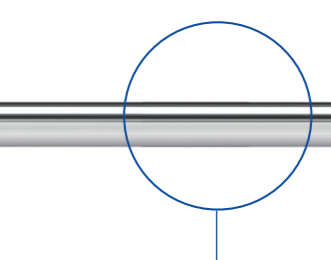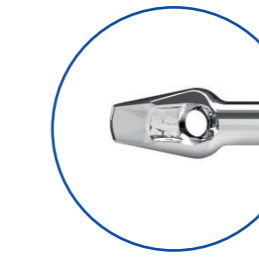

Rigid version without ring marking

HANDLE

- 2 options: longitudinal and lateral alignment
- No unintentional loosening due to lock mechanism
- Feedback at correct adjustment
- Ergonomic design



Original size 1:1



Tunneller

Variations


4 lengths available in standard or rigid version



S - 300 mm


M - 450 mm

L - 600 mm

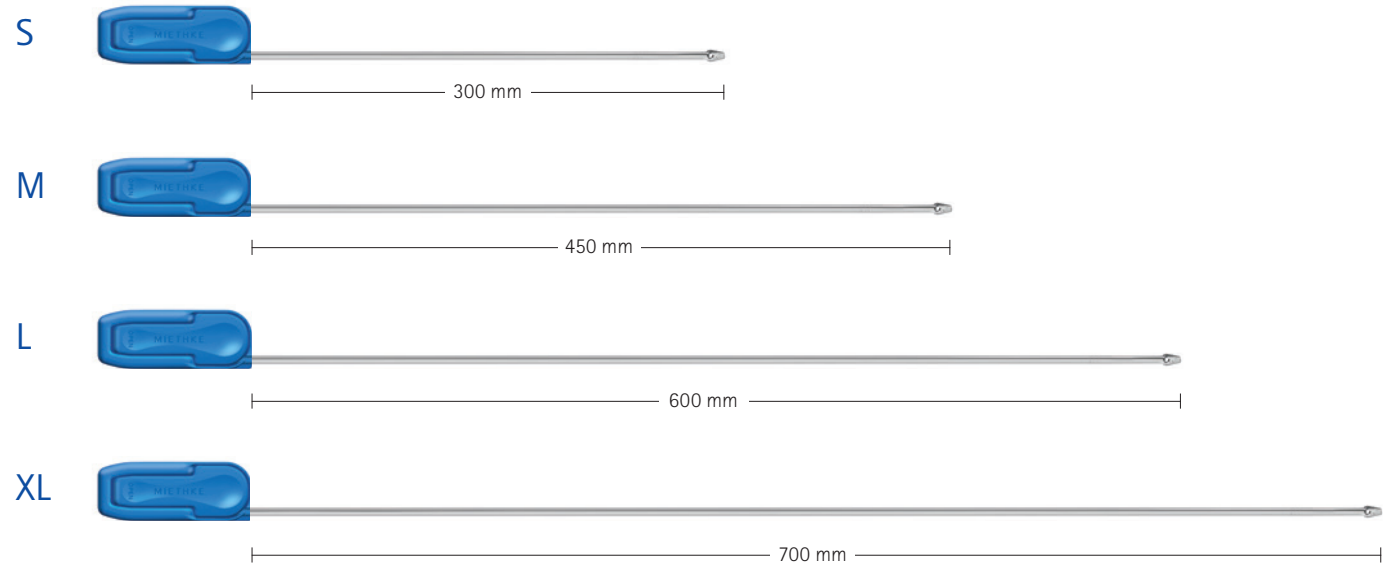XL - 700 mm



	300 mm	450 mm	600 mm	700 mm
 STANDARD VERSION (with ring marking)	FX005SU	FX006SU	FX007SU	FX008SU
 RIGID VERSION (no ring marking)	FX001SU	FX002SU	FX003SU	FX004SU



Through collaborative
excellence we will improve the
quality of a patient's life and
meet the needs of the changing
healthcare environment.

Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogs and price lists and to take legal actions.

©2023 AESCULAP, INC. ALL RIGHTS RESERVED. PRINTED IN THE USA.
Aesculap is an equal opportunity employer

Aesculap, Inc. | 3773 Corporate Parkway | Center Valley, PA | 18034
Phone 800-282-9000 | Fax 610-791-6886 | www.aesculapusa.com

Aesculap, Inc. - a B. Braun company

DOC1790 250 031423