

SHUNTASSISTANT[®] 2.0



Take the headaches out of treating hydrocephalus

Rx Only

INDICATIONS FOR USE: The Miethke Shunt System *SHUNTASSISTANT* (SA 2.0) is used for cerebrospinal fluid (CSF) shunting.



Aesculap Neurosurgery

AESCULAP[®]

SHUNTASSISTANT® 2.0

Take the headaches out of treating hydrocephalus



Overdrainage is one of the most common causes of complications in hydrocephalus treatment.¹⁻³



On average, at least every 5th shunted patient experiences overdrainage.²⁻⁴



Common valve technology does not protect sufficiently against overdrainage symptoms.



Overdrainage can cause severe complications, such as hygromas and hematomas.

- 1 Freimann FB, Sprung C. Shunting with gravitational valves—can adjustments end the era of revisions for overdrainage-related events? *J Neurosurg.* 2012 Dec;117(6):1197-204.
- 2 Lemcke J, Meier U, Müller C, Fritsch MJ, Kehler U, Langer N, Kiefer M, Eymann R, Schuhmann MU, Speil A, Weber F, Remenez V, Rohde V, Ludwig HC, Stengel D. Safety and efficacy of gravitational shunt valves in patients with idiopathic normal pressure hydrocephalus: a pragmatic, randomised, open label, multicentre trial (SVASONA). *J Neurol Neurosurg Psychiatry.* 2013 Aug;84(8):850-7.
- 3 Sundstrom N, Lagebrant M, Eklund A, Koskinen LD, Malm J. Subdural hematomas in 1846 patients with shunted idiopathic normal pressure hydrocephalus: treatment and long-term survival. *J Neurosurg.* 2017 Oct;27:1-8.
- 4 Boon AJ, Tans JT, Delwel EJ, Egeler-Peerdeman SM, Hanlo PW, Wurzer HA, Avezaat CJ, de Jong DA, Gooskens RH, Hermans J. Dutch Normal-Pressure Hydrocephalus Study: randomized comparison of low- and medium-pressure shunts. *J Neurosurg.* 1998 Mar;88(3):490-5.

SHUNTASSISTANT[®] 2.0

The Valve

Gravitational Technology

Depending on the body position of the patient, the SHUNTASSISTANT 2.0 valve gradually adapts the opening pressure automatically and counteracts possible overdrainage.

Combination with Differential Pressure Valves

The SHUNTASSISTANT 2.0 valve proves its strength in combination with other valves. As overdrainage protection, it can be combined with differential pressure valves (fixed or adjustable), either as an initial solution or as a secondary add-on device for patients with existing complications.

Design

The slim, cylindric design enables a fast and easy implantation and is suitable for adults as well as pediatric hydrocephalus treatment.



Additional LP-Variants

SHUNTASSISTANT[®] 2.0 is also suitable for lumbar drainage due to two special variants

SHUNTASSISTANT 2.0 LP Valve
straight





SHUNTASSISTANT 2.0 LP Valve
with deflection, u-form




SHUNTASSISTANT[®] 2.0

Function and Body Position

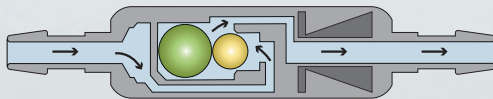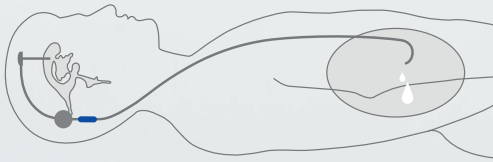


Valve opening pressure
(cmH₂O)

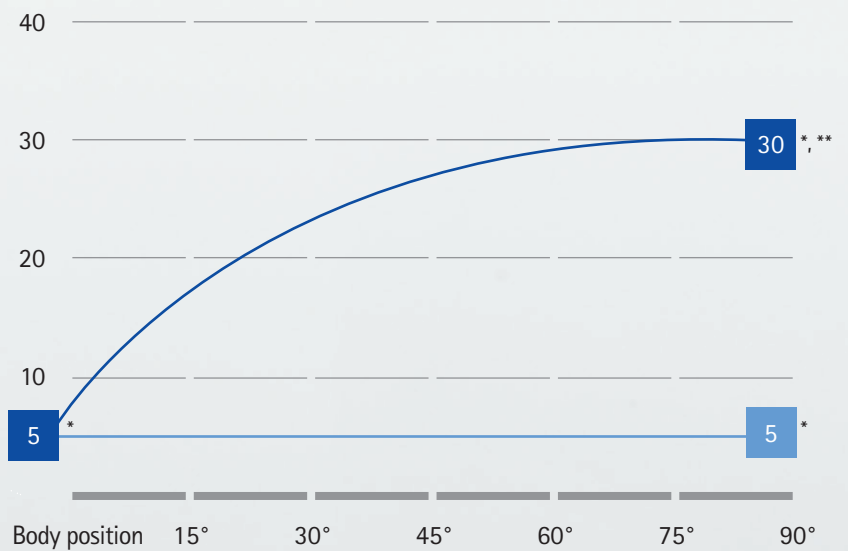

— with the SHUNTASSISTANT 2.0 valve
— without the SHUNTASSISTANT 2.0 valve

* In the example shown, an additional differential pressure valve with opening pressure of 5 cmH₂O, as well as the SHUNTASSISTANT 2.0 valve with 25 cmH₂O have been selected.

** The SHUNTASSISTANT 2.0 valve and the additional differential pressure valve form the total sum of valve opening pressure.

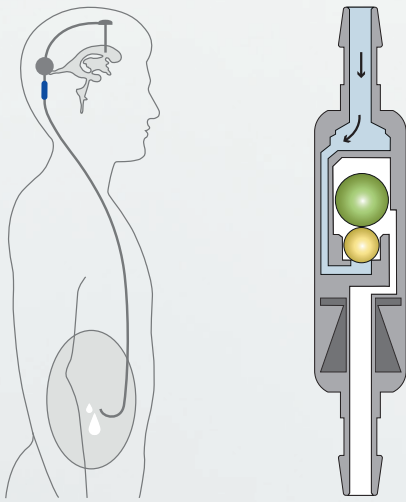

Horizontal Body Position

In the horizontal position, the *SHUNTASSISTANT*[®] 2.0 valve is always open and does not present any resistance. The opening pressure is exclusively determined by an additional differential pressure valve in this body position.

An implantation parallel to the body axis of the patient helps ensure a precise and reliable performance of the *SHUNTASSISTANT* 2.0 valve.

Vertical Body Position

When the patient moves into an upright position, the *SHUNTASSISTANT* 2.0 valve is activated by the tantalum ball (green) and adapts the valve opening pressure automatically.

The functionality of the *SHUNTASSISTANT* 2.0 valve in different body positions is demonstrated in the MIETHKE[®] App.



SHUNTASSISTANT[®] 2.0

X-ray Recognition and Pressure Recommendation

X-ray Recognition


The integrated X-ray coding enables detection of the pressure levels after implantation.

Precision

The valve material titanium assures high precision, durability, reliability and biocompatibility. It helps prevent external and subcutaneous pressure influences and is MR Conditional.


Pressure Levels

The 6 pressure levels offered cover the patient spectrum from newborn to the elderly and allow a wide range of applications in the therapeutic treatment of hydrocephalus.

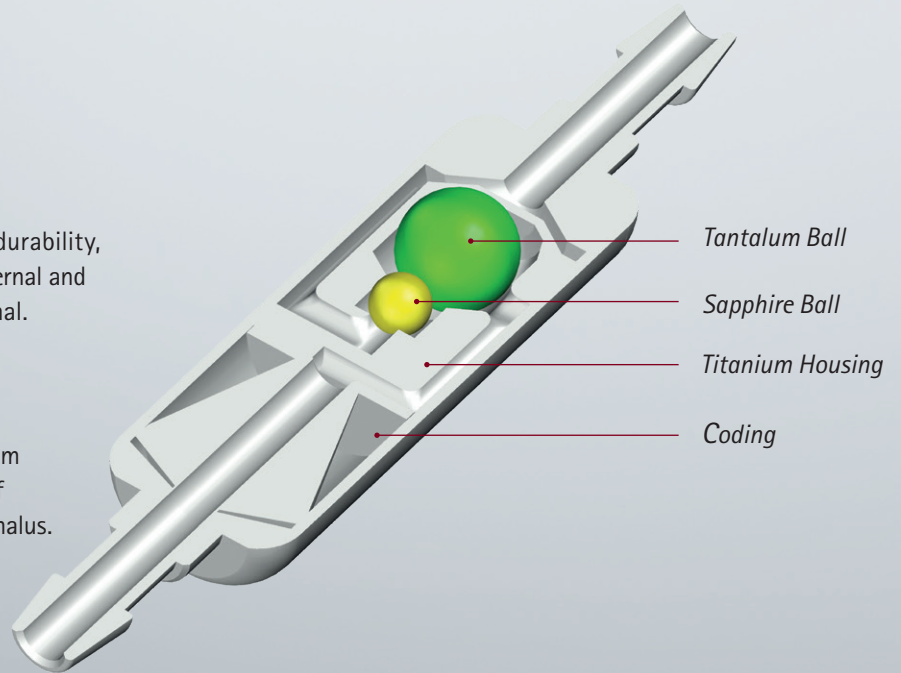

Pressure Level Variants

Pressure Level	X-ray Coding	Radiography Image
10 cmH ₂ O		
15 cmH ₂ O		
20 cmH ₂ O		
25 cmH ₂ O		
30 cmH ₂ O		
35 cmH ₂ O		

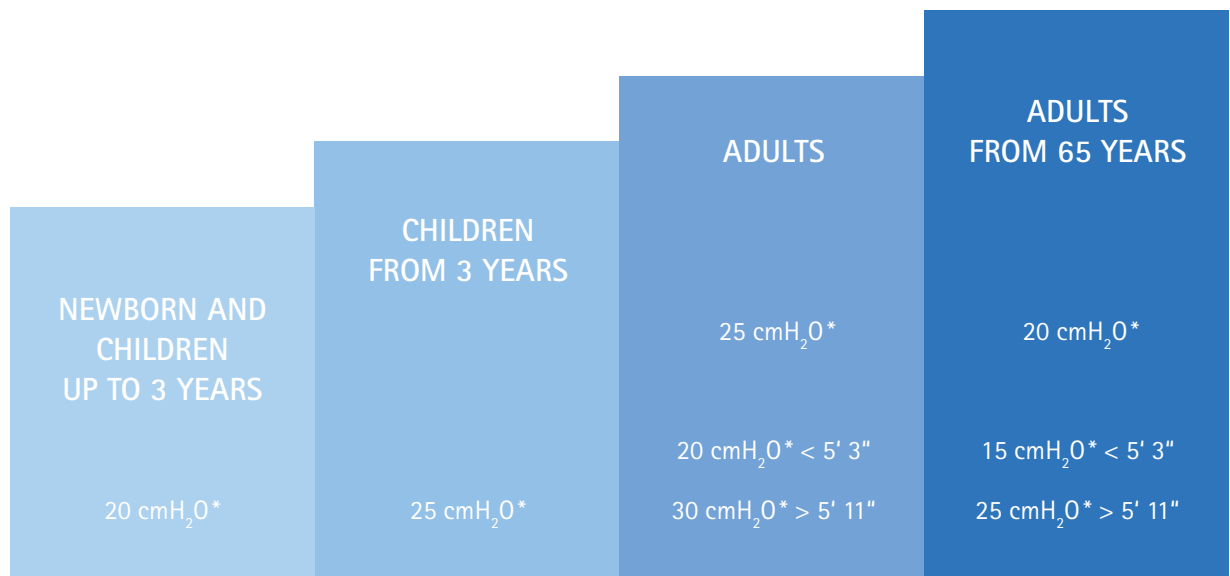

* This is a non-binding recommendation. The treating physician will decide each case individually.

Pressure Recommendation


The choice of the appropriate pressure level of the *SHUNTASSISTANT*[®] 2.0 valve depends on several other factors, including age, degree of activity, size and height of the patient.

The values given apply to mobile patients. For patients with little mobility or a high BMI, a lower pressure level should be chosen for the gravitational unit.

Ordering Information

SHUNTASSISTANT® 2.0 Valve

Valve: $d_o = 4.2$ mm
Connector: $d_o = 1.9$ mm

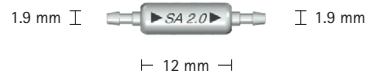


Order	Opening Pressure
-------	------------------

FX100T	10 cmH ₂ O
FX101T	15 cmH ₂ O
FX102T	20 cmH ₂ O
FX103T	25 cmH ₂ O
FX104T	30 cmH ₂ O
FX105T	35 cmH ₂ O

SHUNTASSISTANT 2.0 LP Valve LP, Straight

Valve: $d_o = 4.2$ mm
Connector: $d_o = 1.4$ mm
for connection with lumbar catheter
Connector: $d_o = 1.9$ mm
preferably used with
Catheter: $d_i = 1.2$ mm, $d_o = 2.5$ mm

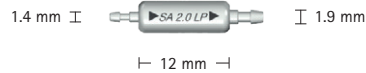


Order	Opening Pressure
-------	------------------

FX106T	10 cmH ₂ O
FX107T	15 cmH ₂ O
FX108T	20 cmH ₂ O
FX109T	25 cmH ₂ O
FX110T	30 cmH ₂ O
FX111T	35 cmH ₂ O

SHUNTASSISTANT 2.0 LP Valve LP, U-form

Valve: $d_o = 4.2$ mm
Connector: $d_o = 1.4$ mm
for connection with lumbar catheter
Connector: $d_o = 1.9$ mm
preferably used with
Catheter: $d_i = 1.2$ mm, $d_o = 2.5$ mm

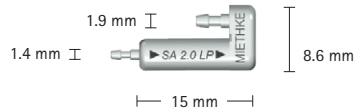


Order	Opening Pressure
-------	------------------

FX112T	10 cmH ₂ O
FX113T	15 cmH ₂ O
FX114T	20 cmH ₂ O
FX115T	25 cmH ₂ O
FX116T	30 cmH ₂ O
FX117T	35 cmH ₂ O

SHUNTASSISTANT® 2.0
Valve with Distal Catheter
(900 mm)

Valve: $d_o = 4.2$ mm
Connector: $d_o = 1.9$ mm
Catheter: $d_i = 1.2$ mm, $d_o = 2.5$ mm

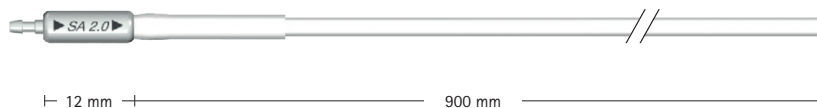

Order	Opening Pressure
FX118T	10 cmH ₂ O
FX119T	15 cmH ₂ O
FX120T	20 cmH ₂ O
FX121T	25 cmH ₂ O
FX122T	30 cmH ₂ O
FX123T	35 cmH ₂ O

SHUNTASSISTANT 2.0 LP
Valve LP, Straight
with Distal Catheter (900 mm)


Valve: $d_o = 4.2$ mm
Connector: $d_o = 1.4$ mm
for connection with lumbar catheter
Connector: $d_o = 1.9$ mm
Catheter: $d_i = 1.2$ mm, $d_o = 2.5$ mm



Order	Opening Pressure
FX124T	10 cmH ₂ O
FX125T	15 cmH ₂ O
FX126T	20 cmH ₂ O
FX127T	25 cmH ₂ O
FX128T	30 cmH ₂ O
FX129T	35 cmH ₂ O

SHUNTASSISTANT 2.0 LP
Valve LP, U-form
with Distal Catheter (900 mm)

Valve: $d_o = 4.2$ mm
Connector: $d_o = 1.4$ mm
for connection with lumbar catheter
Connector: $d_o = 1.9$ mm
Catheter: $d_i = 1.2$ mm, $d_o = 2.5$ mm

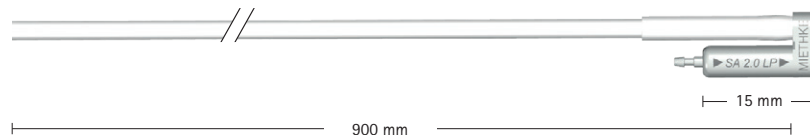

Order	Opening Pressure
FX130T	10 cmH ₂ O
FX131T	15 cmH ₂ O
FX132T	20 cmH ₂ O
FX133T	25 cmH ₂ O
FX134T	30 cmH ₂ O
FX135T	35 cmH ₂ O

All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogs and price lists and to take legal actions.

The product trademarks "proSA", "proGAV", "GAV", "SHUNTASSISTANT", "M.blue", "M.blue plus" and "miniNAV" are registered trademarks of Christoph Miethke GmbH & Co. KG in the majority of the world. For more detailed information please contact us: info@miethke.com. The main product trademark "Aesculap" is a registered trademark of Aesculap AG.

©2020 AESCULAP. ALL RIGHTS RESERVED. PRINTED IN THE USA.
Aesculap is an equal opportunity employer

See Instructions for Use for additional information including Contraindications, Warnings and Precautions.

Aesculap, Inc. | 3773 Corporate Parkway | Center Valley, PA | 18034
Phone 800-282-9000 | Fax 610-791-6886 | www.aesculapusa.com

Aesculap, Inc. - a B. Braun company

DOC1650 Rev A 2M 3/20